

# UPDATED TESTING GUIDANCE

## Abbott BinaxNOW™ COVID-19 Antigen Card

### State Allocation



Kentucky Public Health

Prevent. Promote. Protect.

## PURPOSE

The purpose of this document is to provide guidance for the allocation, distribution, use and reporting of Abbott BinaxNOW™ COVID-19 Antigen (Ag) Card tests in the Commonwealth of Kentucky. The Kentucky Department for Public Health (KDPH) may modify this guidance based upon availability, outbreak surges or other factors during the response to COVID-19.

## BACKGROUND

Through January 2021, the federal government provided Kentucky with BinaxNOW™ COVID-19 Ag Cards. These test cards have been distributed to designated facilities with high-risk, vulnerable populations through coordination with federal, state and local agencies.

The BinaxNOW™ COVID-19 Ag Card is a rapid antigen test with a nasal swab that provides results in as little as 15 minutes. It is intended for use in individuals suspected of COVID-19 by their healthcare provider within the first seven (7) days of symptom onset. This test is a screening tool used in conjunction with established mitigation procedures and does not replace any prevention measures.

## ALLOCATION STRATEGY

Allocations will be based upon expected usage of the BinaxNOW™ COVID-19 Ag Cards including, but not limited to:

- Clinical Settings for Symptomatic Patients
- Outbreak Scenarios
- Surveillance Testing in Settings with Confined Populations

The following types of facilities have been prioritized to request BinaxNOW™ Cards:

- Acute Care Adult Psychiatric Facilities
- Correctional Facilities (State, County, and City)
- Emergency Medical Services (EMS)
- Federally Qualified Health Centers (FQHC)
- Local Health Departments (LHD)
- Residential Substance Use Treatment Centers
- Rural Health Clinics (RHC)
- University/College Health Centers

Beginning January 28, 2021, the following types of facilities are authorized to request BinaxNOW™ Cards:

- All School-Based Clinics (Includes K-12)
- Hospitals (Acute Care and Sub-Acute Care)
- Private Physician/Clinician Offices
- Urgent Treatment Centers

## FACILITY REQUIREMENTS

Use of this authorized test is limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. § 263a, that meet the requirements to perform moderate, high or waived complexity tests and Point of Care (POC) settings operating under a CLIA Certificate of Waiver, Certificate of Compliance or Certificate of Accreditation.

Each facility requesting BinaxNOW™ COVID-19 Ag Cards must meet the following requirements and agree to follow KDPH's policies for use and reporting results:

- Develop a process/plan for using the cards, to include follow up PCR testing as required by KDPH's Antigen Test Result Guidance at:
  - <https://chfs.ky.gov/cv19/AntigenTestResultsGuidance.pdf>
- Train staff to administer the BinaxNOW™ COVID-19 Ag Card test and document this training per agency protocols.
- Report all positive and negative results through established reporting processes.

## TRAINING RESOURCES

### BinaxNOW™ COVID-19 Ag Card

- **Train-the-Trainer:** KDPH will coordinate with Abbott to schedule virtual train-the-trainer courses. Persons will be invited to scheduled classes via established distribution lists.
- **Training:** Abbott has developed videos, modules, guidance documents and FAQs for the BinaxNOW™ COVID-19 Ag test that can be accessed at:
  - <https://www.globalpointofcare.abbott/en/support/productinstallation-training/navica-brand/navica-binaxnow-ag-training.html>

### Kentucky Health Information Exchange (KHIE)

- Training on how to register for an account and submit test reports electronically is available through live trainings, printed materials or online at:
  - <https://khie.ky.gov/COVID-19/Pages/Direct-Lab.aspx>

## QUESTIONS

- CLIA: Contact Truman Taylor, OIG, at [truman.taylor@ky.gov](mailto:truman.taylor@ky.gov)
- BinaxNOW™: 1-800-257-9525 or [ts.scr@abbott.com](mailto:ts.scr@abbott.com)
- KHIE Portal: [KHIElabs@ky.gov](mailto:KHIElabs@ky.gov)
- Allocation/Distribution: [DPH.SEOC@ky.gov](mailto:DPH.SEOC@ky.gov)
- Aggregate Reporting System: [COVIDKYLAB@ky.gov](mailto:COVIDKYLAB@ky.gov)

January 28, 2021

# UPDATED TESTING GUIDANCE

## Abbott BinaxNOW™ COVID-19 Antigen Card

### State Allocation



**Kentucky Public Health**  
Prevent. Promote. Protect.

## REQUESTS & DISTRIBUTION

Facilities can submit requests for the BinaxNOW™ COVID-19 Ag Card tests by completing a Resource Request Form at:

- <https://ky.readyop.com/fs/4iR3/11ea>

KDPH staff will review resource requests and will contact the requestor if additional information is required.

KDPH will distribute tests through UPS to facilities who have been approved through the state's Resource Request Process. Shipments will continue until further notice. LHDs are encouraged to distribute tests throughout their respective jurisdictions in accordance with KDPH guidance.

## STORAGE REQUIREMENTS

Facilities must store test cards between 35.6°F - 86°F. The test cards are stable until the expiration date marked on the outer packaging and containers. All test components must be prepared at room temperature before use.

## SPECIMEN COLLECTION & TESTING

Testing personnel must have a high school diploma, at minimum, and be trained with periodic assessment on their ability to perform quality testing. Due to health and safety concerns, it is recommended that all non-healthcare persons administer the test under the direction of a trained health care professional of the CLIA home office.

Specimens should be tested immediately after collection for optimal test performance. Inadequate specimen collection or improper sample handling/storage/transport may yield erroneous results. Refer to the Centers for Disease Control and Prevention (CDC) for PPE guidance and Interim Guidelines for Collecting, Handling and Testing Clinical Specimens from Persons for Coronavirus Disease 2019 (COVID-19) at:

- <https://www.cdc.gov/coronavirus/2019-ncov/lab/guidelines-clinical-specimens.html>

## DISPOSAL

ALL components of this COVID-19 antigen test should be discarded as biohazard waste according to federal, state and local regulatory requirements.

## RESULT INTERPRETATION

- **Antigen Guidance:** Refer to KDPH's Antigen Test Results Guidance "Response to Point-of-Care Antigen Test Results" at:

- <https://chfs.ky.gov/cv19/AntigenTestResultsGuidance.pdf>

- **Negative Result:** A negative specimen will give a single pink/purple colored *Control Line* in the top half of the window, indicating a negative result. This *Control Line* means that the detection part of the test was performed correctly, but no COVID-19 antigen was detected.

### Negative Result




- **Positive Result:** A positive specimen will give two pink/purple colored lines. This means that COVID-19 antigen was detected. Specimens with low levels of antigen may give a faint *Sample Line*. Any visible pink/purple colored *Sample Line* designates a positive result.

### Positive Result



- **Invalid Result:** If no control and no sample lines are seen, if just the sample line is seen, or the *Blue Control Line* remains blue, the assay is invalid. Invalid tests should be repeated.

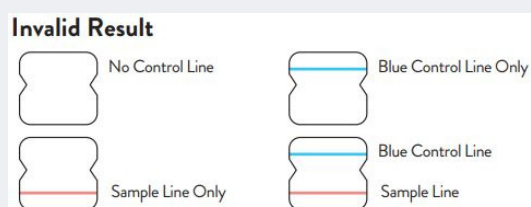

## REPORTING REQUIREMENTS

All laboratories and facilities performing COVID-19 testing must report results within 24 hours following KDPH's COVID-19 Case Reporting Guidance at:

- COVID-19 Lab Result Reporting Guidance
  - <https://chfs.ky.gov/agencies/dph/covid19/ClinicalCommercialLabCovidGuidance.pdf>
- Healthcare Provider Case Reporting Guidance
  - <https://chfs.ky.gov/agencies/dph/covid19/HealthcareProviderGuidance.pdf>

**NOTE:** For *mass testing situations* with state-allocated BinaxNOW™ Cards, facilities should follow the attached "Lab Result Reporting for State-Supplied BinaxNOW™ Antigen Test Cards" guidance.

## QUESTIONS

- CLIA: Contact Truman Taylor, OIG, at [truman.taylor@ky.gov](mailto:truman.taylor@ky.gov)
- BinaxNOW™: 1-800-257-9525 or [ts.scr@abbott.com](mailto:ts.scr@abbott.com)
- KHIE Portal: [KHIElabs@ky.gov](mailto:KHIElabs@ky.gov)
- Allocation/Distribution: [DPH.SEOC@ky.gov](mailto:DPH.SEOC@ky.gov)
- Aggregate Reporting System: [COVIDKYLAB@ky.gov](mailto:COVIDKYLAB@ky.gov)